
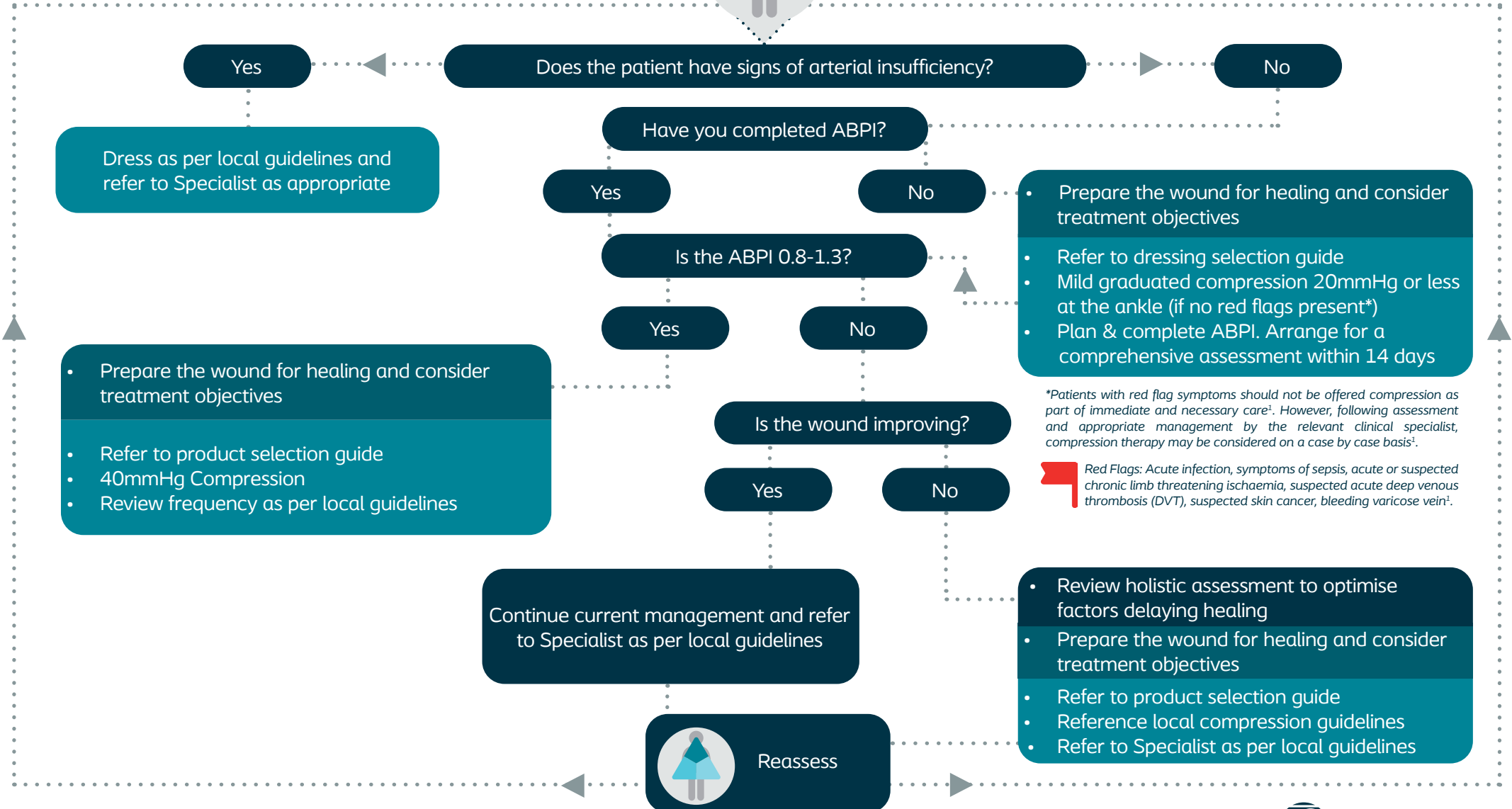


Simplifying wound healing Venous Leg Ulcer Pathway

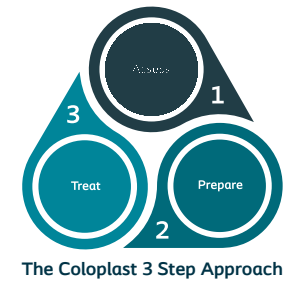

*Patients with red flag symptoms should not be offered compression as part of immediate and necessary care¹. However, following assessment and appropriate management by the relevant clinical specialist, compression therapy may be considered on a case by case basis¹.

Red Flags: Acute infection, symptoms of sepsis, acute or suspected chronic limb threatening ischaemia, suspected acute deep venous thrombosis (DVT), suspected skin cancer, bleeding varicose vein¹.

Simplifying product choice

Venous Leg Ulcer Product Selection Guide



Step 1: Assess

Assess the wound using the Venous Leg Ulcer pathway

Step 2: Prepare

Prepare the wound for healing.
Use Alprep® Pad or follow local guidance.

Step 3: Treat

Treat the wound in accordance with underlying aetiology and specific wound treatment objectives

Alprep® Pad



Compression Hosiery /
Leg Wraps

Compression
Bandaging

No infection present



Biatain® Silicone
or Biatain® Silicone Lite



Biatain® Silicone Non-Border
or Biatain® Non-Adhesive

Infection present



Biatain® Silicone Ag



Biatain® Ag Non-Adhesive

Patient experiencing pain



Biatain® Ibu



Biatain® Ibu

If high levels of exudate consider including **Biatain® Fiber**.

If a contact layer is required consider including **Biatain® Contact**.



If pain is experienced by the patient and thought to be caused by infection, make treatment of infection the priority whilst managing the pain and treating the underlying venous disease.