


Simplifying wound care for you

Pressure Ulcer Pathway

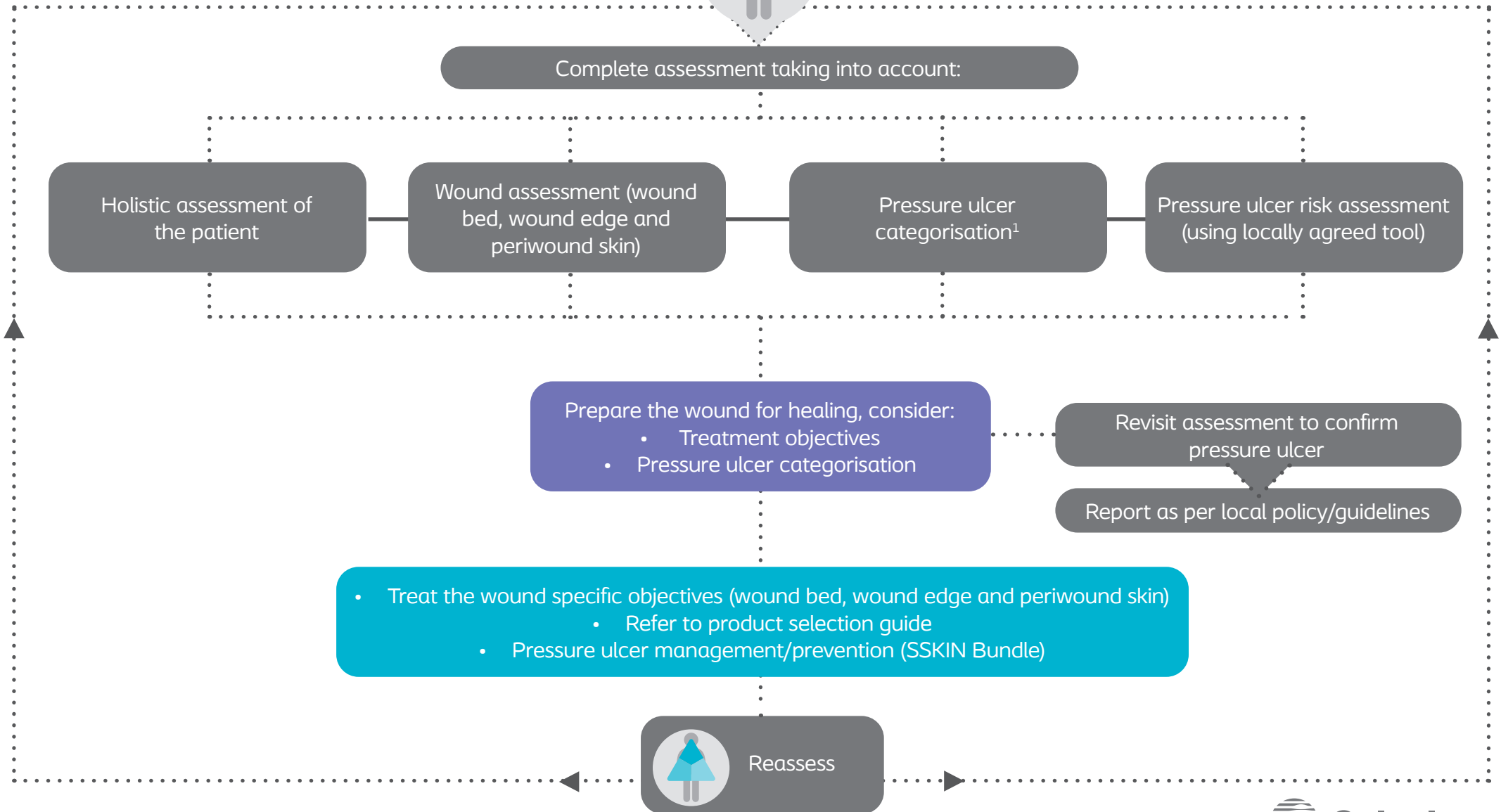

The Coloplast 3 Step Approach



Simplifying wound care for you

Pressure Ulcer Product Selection Guide



Step 1: Assess

Assess the wound using the Pressure Ulcer pathway

Step 2: Prepare

Prepare the wound for healing.
Use Alprep® Pad or follow local guidance.


Step 3: Treat

Treat the wound in accordance with underlying aetiology and specific wound treatment objectives

Identify wound treatment goal:

Protect wound bed,
wound edge &
periwound skin


Comfeel® Plus Range

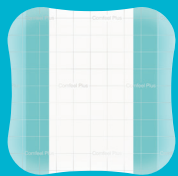

Also consider
Biatain® Silicone Lite

Rehydrate the wound

Comfeel® Plus Range

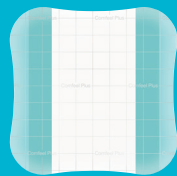

Also consider
Biatain® Silicone Lite

Manage exudate

0-2cm in depth*



Biatain® Silicone

>2cm in depth
and/or undermining/tunneling



Biatain® Fiber
+ **Biatain® Foam**

Remove devitalised tissue

Biatain® Silicone



Also consider
Biatain® Fiber

Manage bacterial burden

0-2cm in depth*



Biatain® Ag Foam

>2cm in depth
and/or undermining/tunneling



Biatain® Alginate Ag
+ **Biatain® Foam**

*Conformability may vary across product design

This information is for general guidance only and should not replace clinical judgement. If you are unsure, seek TVN advice

The Coloplast logo is a registered trademark of Coloplast A/S. © 2021 All rights reserved Coloplast A/S

CM775N