


Simplifying wound care for you

Venous Leg Ulcer Pathway

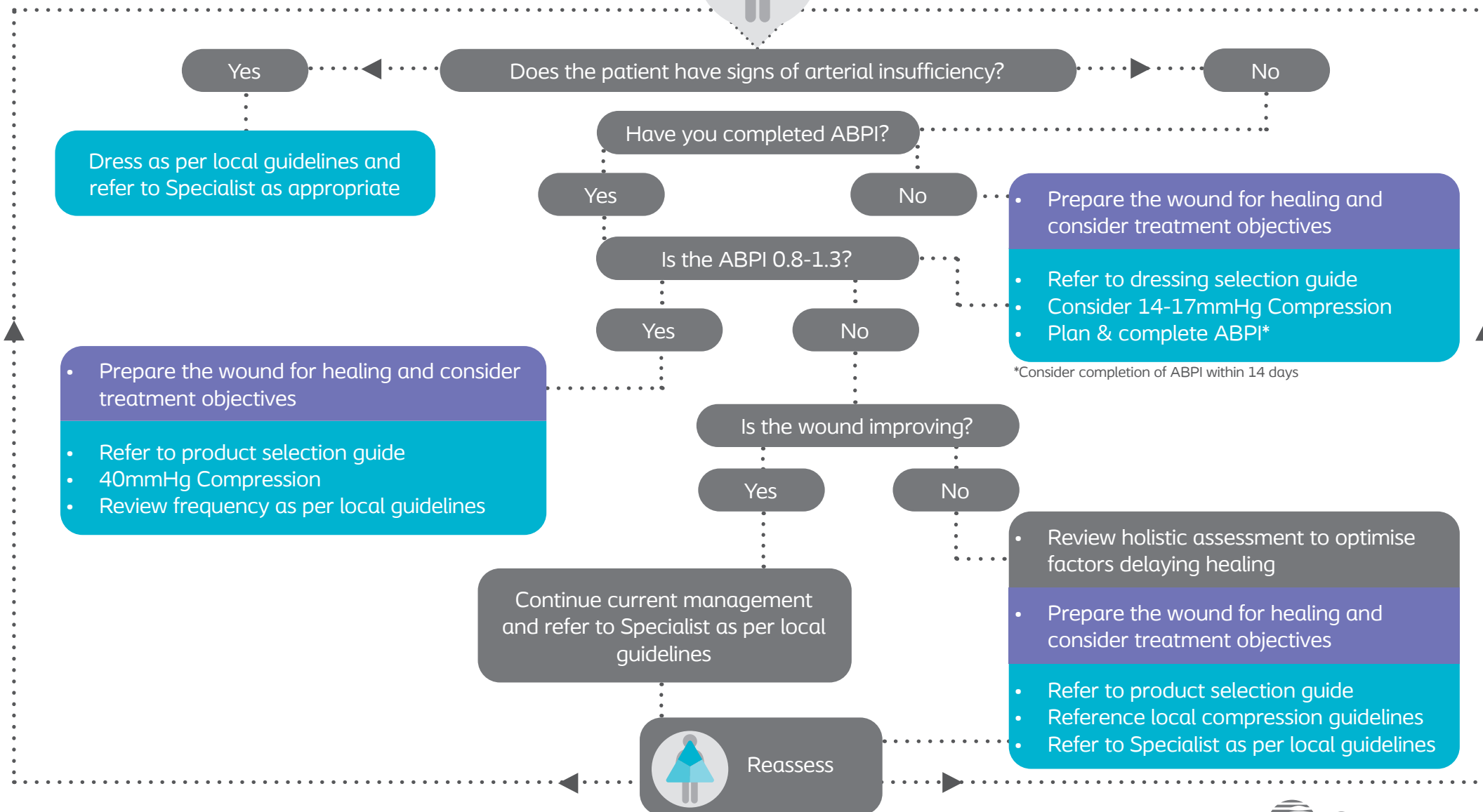

The Coloplast 3 Step Approach



Simplifying wound care for you

Venous Leg Ulcer Product Selection Guide



Step 1: Assess

Assess the wound using the Venous Leg Ulcer pathway

Step 2: Prepare

Prepare the wound for healing.
Use Alprep® Pad or follow local guidance.

Step 3: Treat

Treat the wound in accordance with underlying aetiology and specific wound treatment objectives

Compression Hosiery /
Leg Wraps

Compression
Bandaging

No infection present



Biatain® Silicone
Biatain® Silicone Lite



Biatain® Non-Adhesive
Biatain® Soft-Hold

Infection present



Biatain® Silicone Ag



Biatain® Ag Non-Adhesive

Patient experiencing pain



Biatain® Ibu



Biatain® Ibu

If high levels of exudate consider including **Biatain® Fiber**.

If a contact layer is required consider including **Biatain® Contact**.



If pain is experienced by the patient and thought to be caused by infection, make treatment of infection the priority whilst managing the pain and treating the underlying venous disease.