

The same Peristeen[®] you know. Now easier to use!




New connections
are easier to use
and handle




New symbols on the control unit
are more intuitive and
easier to understand



Water temperature indicator
helps to identify the correct
temperature for irrigating

 Peristeen Plus is also available without the toiletry bag to help reduce our environmental footprint

Peristeen Plus Transanal Irrigation Conversion Chart

 **Important information:** Catheters and tubing from the old Peristeen system are not compatible with the new Peristeen Plus system





Current Peristeen Order Code	New Peristeen Plus Order Code	Description	Contents
29121	29141	Peristeen Plus system Regular (excl toiletry bag)	1 control unit, 2 rectal catheters Regular , 1 water bag (incl screw top), 2 straps, 1 tube
29122	29142	Peristeen Plus accessory unit Regular	15 catheters Regular , 1 water bag
29126	29148	Peristeen Plus system Small (excl toiletry bag)	1 control unit, 2 rectal catheters Small , 1 water bag (incl screw top), 2 straps, 1 tube
29127	29149	Peristeen Plus accessory unit Small	15 catheters Small , 1 water bag
Additional Products			
29121	29140	Peristeen Plus system Regular incl toiletry bag (if new patient)	1 control unit, 2 rectal catheters Regular , 1 water bag (incl screw top), 2 straps, 1 tube, 1 toiletry bag
29126	29147	Peristeen Plus system Small incl toiletry bag (if new patient)	1 control unit, 2 rectal catheters Small , 1 water bag (incl screw top), 2 straps, 1 tube, 1 toiletry bag
29124	29145	Peristeen Plus strap	10 sets of 2 straps
29125	29146	Peristeen Plus tube	2 tubes with turquoise connectors

Peristeen[®] Plus

Getting to know the Peristeen Plus system

- 1. Screw top (including lid)**
Replace after 90 uses
Remember to keep the lid when you change the water bag
- 2. Water bag including temperature indicator**
Replace after 15 uses
- 3. Control unit**
Replace after 90 uses
- 3a. Pump for inflating balloon and pumping water**
- 3b. Dial for regulating air and water**
- 4. Tubes**
Replace after 90 uses
- 5. Coated rectal balloon catheters**
Single-use only
- 6. Straps**
Replace when elasticity weakens
- 7. Toiletry bag***
For system storage

Control unit symbols

- | | |
|--|---|
|  Turquoise water symbol |  Grey deflate symbol |
|  White balloon symbol |  Finish and storage symbol |



* Peristeen Plus is also available without the toiletry bag to help reduce our environmental footprint

CV1055N



Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

Coloplast Ltd, Nene Hall, Peterborough Business Park, Peterborough PE2 6FX

www.coloplast.co.uk The Coloplast logo is a registered trademark of Coloplast A/S © 2021-09 All rights reserved Coloplast A/S PM-18546

 **Coloplast**